

### LED light & holder



Thank you for choosing the 1000 Lumen Trionic LED Front Light.

This manual both shows how you attach the light holder and the light on a Walker (left column below) and also how you attach them on a Veloped (right column below).

In the left-hand photo, **A** shows the holder attachment parts for a Walker (2 screws + 2 washers). **B** shows the holder attachment parts for a Veloped (1 screw + 1 knurled nut + 1 nut).

## Attaching the Light Holder

### Instructions for a Walker



**A1.** On top of the black semi-vertical tube, locate the two screws that clamp the grip tube, situated in front of the brake lever.



### Instructions for a Veloped



**A1.** Locate the screw at the center folding hinge of the U-shaped grip bar of the Veloped. Unscrew the screw with the included 4mm allen key and remove it.



# Attaching the LED Light to the Holder

---

## Instructions for a Walker



**A6.** Identify the circular front end of the light holder, with two slots for attaching the light.




**A7.** Push the slotted bracket of the light into the slots of the holder and make sure they are aligned. Seen in the traveling direction of the Walker, enter the included wingnut from the left and tighten it.

## Instructions for a Veloped



**B6.** Locate the screw at the center folding hinge of the U-shaped grip bar of the Veloped. Unscrew the screw with the included 4mm allen key and remove it.

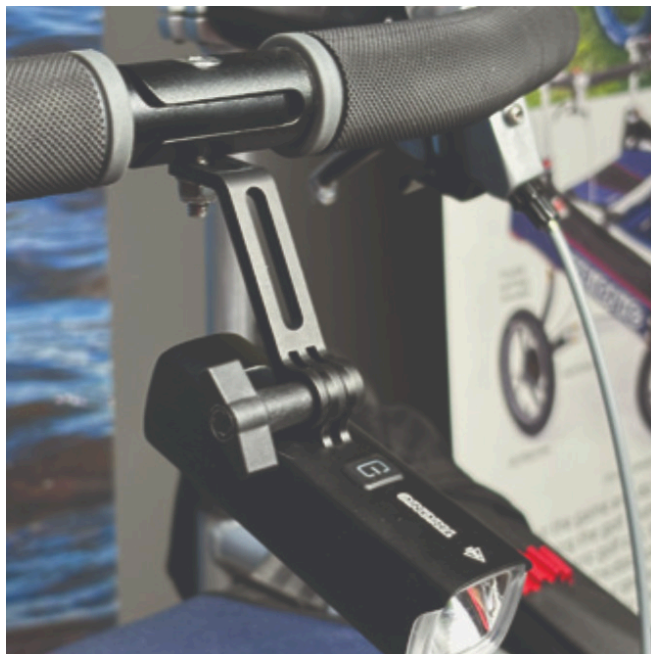

**B7.** Identify the circular front end of the light holder, with two slots for attaching the light.

## Instructions for Use

---

### Overview